
NITRAX DIAMOND
Model: X2000

PL	Rękawice ochronne	RU	Заштитні перчатки
EN	Protective gloves	HR	Zaštitne rukavice
DE	Schutzhandschuhe	HU	Védőkesztyű
CZ	Ochranné rukavice	RO	Mănuși de Protecție
SK	Ochranné rukavice	IT	Guanti di Protezione
LT	Apsaugines pirštines	EE	Kaitsekindad

EN ISO 374-1:2016 +A1:2018/TYP B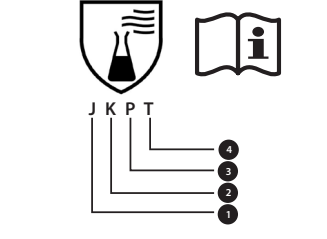

EN ISO 374-1:2016+A1:2018/TYPE B	EN ISO 374-4:2019	poziom / level / Niveau / úroveň / уroveň / razina / szint / nivel / livello	degradacja / Degradation / degradace / degradácia / degradation / degradació / degradare / degradazione
1	6	44.3 %	n-Heptane / n-Heptan / n-heptan / n-heptán / n-heptanas / n-Heptan / n-heptan / n-heptán / heptan / eptano (J)
2	5	10.0%	40% wodorotlenek sodu / 40% Sodium Hydroxide / 40% Natronlauge / 40% hydroxid sodný / 40% hidroxid sodný / 40% natrio hidrossidas / 40% natrio idrossido / 40% natrio idrossido di sodio / 40% natrijev hidroksid / 40% natrium-hydroxid / 40% hidroxid de sodiu / 40% idrossido di sodio (K)
3	2	36.5 %	30% nadtlenek wodoru / 30% Hydrogen Peroxide / 30% Wasserstoffperoxid / 30% peroxid vodíku / 30% peroxid vodik / 30% peroxid vandenilio / 30% peroksidás / 30% перекись водороду / 30% vodikov peroksid / 30% hidrogen-peroksid / 30% peroxid de hidrogen / 30% perossido di idrogeno (P)
4	5	17.5%	37% formaldehyd / 37% Formaldehyde / 37% Formaldehyd / 37% formaldehid / 37% formaldehid / 37% formaldehid / 37% formaldehid / 37% formaldehid / 37% formaldehid (T)

czas przebiega wyznaczony w badaniach laboratoryjnych / breakthrough time determined in laboratory tests / Durchbruchzeit, die in Laboruntersuchungen bestimmt wurde / čas proraženi úroveň v laboratorních testech / čas prirazu určeny v laboratorných testoch / prověřeno laikas, nustatyta laboratoriniuose tyrimuose / время пробы, определенное в лабораторных испытаниях / vrijeme probanja određeno u laboratorijskim ispitivanjima / áthatalási idő, ameyelt laboratóriumi vizsgálatok határozatk meg / timpul de spargere stabilit în teste de laborator / läbimurdeag, mis määratakse laboratoorses katsetes [mm]	<p>Poziom skuteczność odporności na przenikanie / Level of resistance to penetration effectiveness / Niveau der Widerstandsfähigkeit gegen Eindringen / Úroveň odolnosti proti pronikání / Úroveň odolnosti proti prieniku / Atsparumo lygis praskiverimui / Уровень устойчивости к проникновению / razina otpornosti na prodiranje / Ellenállás a behatolás ellen / Nivelul de rezistență la penetrare / Läbimisvastupidavuse tase</p>
>10	1
>30	2
>60	3
>120	4
>240	5
>480	6

EN ISO 374-5:2016

VIRUS

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EN
Size range: 8-11

The gloves belong to category III of personal protection measures. Disposable gloves meet the essential requirements of the Regulation of the European Parliament and of the Council (EU) 2016/425 on personal protective equipment, the requirements of EN ISO 21420:2020, EN ISO 374-1:2016+A1:2018, EN ISO 374-5:2016 and the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices: class I Notified body involved in conformity assessment: SATRA Technology Europe Ltd., Bracetown Business Park Clonee, Dublin, D15 YN2P; Ireland Notified body number: 2777 Declaration of conformity available at www.stalco.pl

Intended use: domestic work, workshop, cleaning
Performance and limitations of use - This product has been tested and achieved the following performance levels: see tables and labelling

Instructions and restrictions on use (protection):
EN ISO 374-4:2019 Degradation results show the change in puncture resistance of the glove after exposure to the test chemical.

EN ISO 374-5:2016 The resistance to penetration has been assessed under laboratory conditions and applies only to the specimen under test.

This information does not reflect the actual duration of protection in the workplace and the distinction between mixtures and pure chemicals.

The chemical resistance was assessed under laboratory conditions using specimens taken only from the hand (except where the glove is 400 mm or more in length - where the cuff is also tested) and relates only to the chemical being tested. This may be different if the chemical is used in a mixture.

It is advisable to check that the gloves are suitable for the intended use, as work conditions may differ from the type test depending on temperature, abrasion and degradation.

During use, protective gloves may provide less resistance to a hazardous chemical due to changes in physical properties. Movements, snags, friction, degradation due to contact with chemicals, etc. can significantly reduce the actual service life. For corrosive chemicals, degradation may be the most important factor to consider when choosing chemical-resistant gloves.

The gloves should be checked for any defects or damage before use.

The glove performance quoted is based on laboratory data and may not reflect actual protection time in the workplace due to other factors affecting performance, such as temperature, abrasion, degradation, etc.).

The gloves do not contain any substances known to cause allergies. The gloves do not provide mechanical protection. For single use only, do not litter. Check gloves for damage before use.

Establishment:
1. Remove all jewellery from your hands and wrists and wash your hands before putting it on.
2. Place the gloves on the prepared work surface.
3. The user places the glove on his or her dominant hand, grasping it with the other hand, making sure to touch only the inside of the glove, and slipping it onto the dominant hand until it reaches searcher level.
4. The user uses the dominant gloved hand to slide the other glove onto the non-dominant hand.
5. Once both gloves are on, users can touch the outside of the glove to ensure a proper fit.

Removal:
1. Using the dominant hand, users start by gripping the outside of the glove on the non-dominant hand on the palm side near the cuff.
2. Remove the glove from the non-dominant hand and place it in the gloved hand, rolling it into a ball.
3. Slide two fingers under the cuff of the second glove and carefully remove it from the hand without touching the wrist, turning the remaining glove inward as you remove it and in turn embracing the first glove.
4. Gloves can be disposed of.

Storage and transport
Store in original packaging, at room temperature, normal humidity, in closed and ventilated areas, protected from UV rays and moisture, away from heat sources.

Transport in original packaging. During transport, protect the packaging from damage and environmental factors.

The expected shelf life of the gloves is 4 years if stored under appropriate conditions (humidity, temperature, cleanliness, ventilation, light).
Cleaning and maintenance: Do not wash. Single-use gloves. Once used, they should be disposed of.
Disposal: Gloves in use may be contaminated with infectious and/or hazardous substances. Dispose of in accordance with local country regulations.
Caution: Read the contents of this user guide before use. It is advisable to keep the instructions.

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